

# TruCric

## User Manual



***An innovative and durable solution for practicing realistic front of neck access techniques***

*The TruCric model features a unique larynx insert facilitating a lifelike training experience for users practising needle/surgical cricothyroidotomy and percutaneous tracheostomy. Easily interchangeable and cost-effective consumables make this model ideal for a busy teaching environment.*

## Contents

Product specifications .....	3
Package contents .....	3
Initial set-up information .....	3
List of procedures facilitated by the TruCric .....	4
Replacement of consumables.....	4
Care and maintenance .....	5
Warranty.....	5

## Product specifications

**Product Code:** TCRIC1

**TruCric weight:** Approx. 2kg

**Full shipment weight:** Approx. 2.1kg

**Full shipment dimensions:** 40 x 24 x 22cm

## Package contents

- 1 TruCric ready-to-use (TCRIC1)
- 1 TruCric carrier case (CBAG03)
- 100ml bottle of lubrication (TL001)
- 10 Larynx inserts (NLX050 – 1 supplied on the manikin and 9 spare)
- 2 wraparound neck skin (RSN1005 – 1 supplied on manikin and 1 spare)
- 1 non-slip mat to place under model during training

## Initial set-up information

- Gently remove the model from the carrier case
- Secure the model to a suitable surface and ensure the non-slip mat is placed underneath
- Please ensure the lung bag is attached to the airway tube
- Ensure the larynx is in correct position with the wraparound skin secured in place
- Spray generous amounts of lubrication provided with the model (TL001) on equipment prior to use

## List of procedures facilitated by the TruCric

- Needle and surgical cricothyroidotomy
- Percutaneous tracheostomy
- Air Jet ventilation

## Replacement of consumables

The larynx insert is designed to be replaced for each student to provide a unique training experience. This is a very quick and easy process, perfect for a busy teaching environment.

1. Remove the wraparound neck skin to provide access to the larynx insert
2. Remove the larynx insert and discard



3. Align a new insert into the larynx holder and press into place. Please ensure the ridges at all corners of the insert are tightly aligned with the larynx insert frame:



4. When replacing the neck skin ensure that it passes through the pre-defined gaps on the base, and the skin rests on the larynx insert to give prominent definition

## Care and maintenance

Store in clean, dry conditions away from heat and direct sunlight; avoid contact with metals, solvents, oils or greases and strong detergents. When the product is not in use please store in the black carrier case provided.

Thoroughly wash the AirSim X airway in warm water. Please use warm soapy water or something similar until all visible foreign matter and residue is removed.

Mild detergents or enzymatic cleaning agents may be used on the airway in accordance with the manufacturer's instructions and at the proper dilution. The detergent must not contain skin or mucous membrane irritants.

*Please do not use any of the following when cleaning the AirSim product range*

- Germicides, disinfectants, or chemical agents such as glutaraldehyde (e.g. Cidex®),
- Ethylene oxide, phenol-based cleaners or iodine-containing cleaners

**In response to the recent COVID-19 pandemic, we recommend this additional step to ensure the product is fully sanitised:**

Use alcohol spray (minimum 75%) and wipe off. Repeat this for 3-4 times to ensure to kill the virus completely.

## Warranty

TruCorp warrants this unit to be free of defects in materials and workmanship and to give satisfactory service for a period of 1-year from the date of delivery. This ensures that our customers receive maximum coverage on each product. If the unit should malfunction it must be returned to the factory for evaluation. Upon examination by TruCorp, if the unit is found to be defective it will be repaired or replaced at no charge.

TruCorp will pay for the freight/delivery and the actual parts needed free of charge if any part of the product fails within the 1-year period.

However, these warranties are VOID, if; the unit shows evidence of having been tampered with or shows evidence of having been damaged by excessive heat, the use of sharp instruments, misapplication, misuse or other operating conditions outside of TruCorp's control. Components which wear or are damaged by misuse are not warranted and will be charged for if repair has been approved. Warranty is void if third party products are seen to have damaged or caused failure of the TruCorp models.

**Please direct all warranty and repair inquiries to:**



**TruCorp Ltd**  
**33 Waringstown Road**  
**Lurgan, Co. Armagh**  
**BT66 7HH**  
**Northern Ireland**  
**[info@trucorp.com](mailto:info@trucorp.com)**  
**Tel: +44 (0) 28 3888 2714**